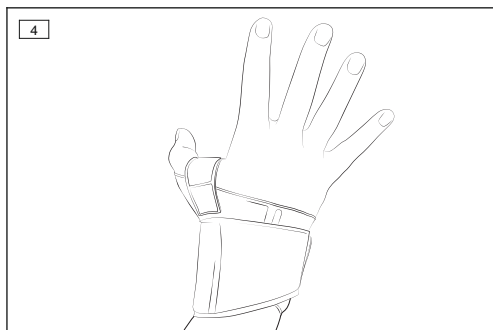
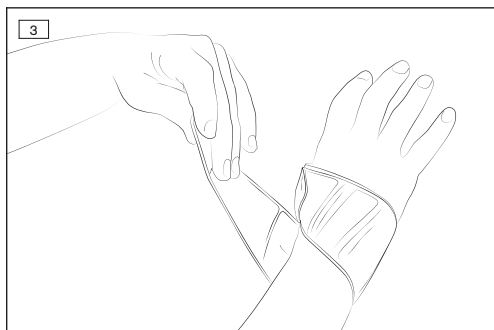
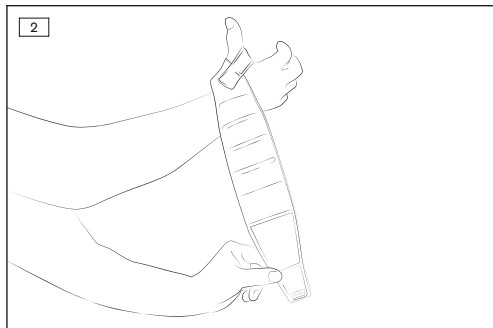
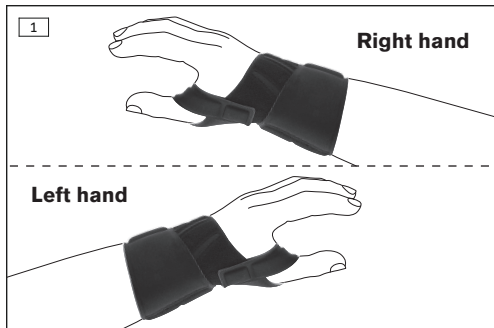
## 50P70 Manu Carezza

<b>DE</b> Gebrauchsanweisung .....	4	<b>NO</b> Bruksanvisning .....	26
<b>EN</b> Instructions for use .....	6	<b>FI</b> Käyttöohje .....	29
<b>FR</b> Instructions d'utilisation .....	9	<b>PL</b> Instrukcja użytkowania .....	31
<b>IT</b> Istruzioni per l'uso .....	11	<b>HU</b> Használati utasítás .....	34
<b>ES</b> Instrucciones de uso .....	14	<b>CS</b> Návod k použití .....	36
<b>PT</b> Manual de utilização .....	16	<b>SK</b> Návod na používanie .....	39
<b>NL</b> Gebruiksaanwijzing .....	19	<b>RU</b> Руководство по применению .....	41
<b>SV</b> Bruksanvisning .....	21	<b>JA</b> 取扱説明書 .....	44
<b>DA</b> Brugsanvisning .....	24	<b>ZH</b> 使用说明书 .....	46

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### INFORMATION

Last update: 2018-05-22

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide important information on the application of the 50P70 Manu Carezza wrist support.

## 2 Intended use

### 2.1 Indications for use

The support is intended **exclusively** for treating the upper limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

### 2.2 Indications

- Wrist instability
- Scaphoid pseudoarthrosis
- Mild irritation and instability

The indication must be determined by the physician.

## 2.3 Contraindications

### 2.3.1 Absolute Contraindications

None known.

### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases / injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the device will be applied; sensory and circulatory disorders in the body region being treated and in the area of the fingers.

## 2.4 Mechanism of Action

The support braces the wrist joint and relieves pain.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION**

Warning regarding possible risks of accident or injury.

 **NOTICE**

Warning regarding possible technical damage.

### 3.2 General safety instructions

 **CAUTION**

#### Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

 **CAUTION**

#### Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

 **NOTICE**

#### Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

 **NOTICE**

#### Improper molding or application

Damage to the product due to overloading of the material and improper fit of the product due to breakage of load-bearing components

- ▶ The product may only be molded by qualified personnel.
- ▶ Do not make any improper changes to the product.
- ▶ Always apply the product according to the information in the instructions.

 **NOTICE**

#### Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

## 4 Handling

 **INFORMATION**

- ▶ The daily duration of use and period of application are generally determined by the physician.

- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

#### 4.1 Size selection

The support is available in one universal size and can be worn on either side. The maximum wrist circumference is **22 cm**.

#### 4.2 Fitting and Application

##### **CAUTION**

##### **Incorrect or excessively tight application**

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

The support is pre-configured for use on the right hand upon delivery. To use it on the left hand, change the direction of the thumb closure loop on the support (see fig. 1).

- 1) Unroll the support; the hook-and-loop closure for the thumb remains fastened.
  - 2) Guide the thumb through the thumb opening (see fig. 2).
  - 3) Guide the wide hook-and-loop closure of the support over the back of the hand and fasten it at the wrist (see fig. 3).
  - 4) Check the fit of the support and use the hook-and-loop closures to optimize the fit of the support if desired (see fig. 4).
- The support fits snugly without pinching.

#### 4.3 Removal

- 1) Open the hook-and-loop closure of the support; the hook-and-loop closure for the thumb remains fastened.
- 2) Remove the support.

## 5 Cleaning

##### **NOTICE**

##### **Use of improper cleaning agents**

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Fasten all hook-and-loop closures.
- 2) Recommendation: Use a laundry bag or net.
- 3) Wash the support in warm water at **40 °C (104 °F)** with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 6 Disposal

Dispose of the product in accordance with national regulations.

## 7 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

### 7.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The manufacturer therefore drew up the declaration of conformity on its own responsibility in accordance with Annex VII of the directive.

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